

PROLITE® BI-LATERAL ANKLE BRACE

Model No: 40-351


**Sizes: Small - XLarge
Color: White**

INDICATIONS:

- Instability
- Moderate Sprains
- Post Surgery Ankle Weakness

BENEFITS:

This lightweight lace-up ankle brace provides ankle stability to weak or injured ankles. Made of a soft, pliable material that molds to the foot and ankle for a tight fit. Inner elastic insets increase the conforming support to simulate taping of the ankle. Metal spring stays are anatomically shaped to provide stability and support to the medial and lateral ligaments of the ankle. Elastic insets at the heel and metatarsal areas provide support during movement and prevent pinching at the Achilles Tendon and the arch of the foot.

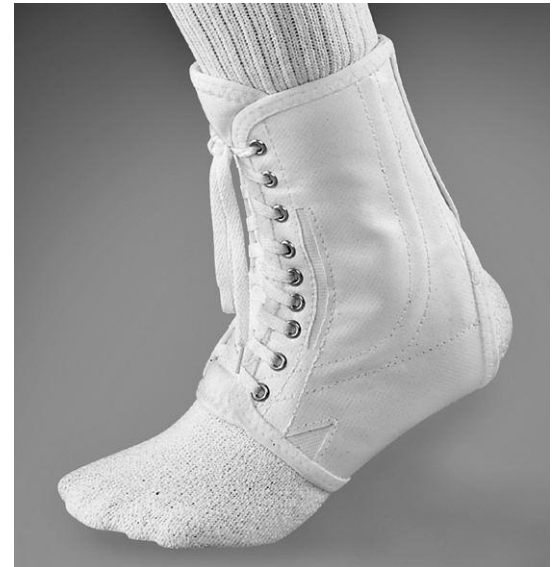

FEATURES:

- Flexible J-shaped metal spring stays stabilize both sides of the ankle
- Soft, pliable material for a conforming fit
- Elastic Insets for a conforming fit
- Low profile design easily fits in most shoes
- Fits left or right

HOW TO APPLY:

Wear brace over socks. Slip foot into brace and tighten laces from bottom to top. Retighten for an even and comfortable fit.

CARE INSTRUCTIONS:

Machine wash gentle cycle or hand wash in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat. Do not bleach.

SIZING:

Size according to shoe size.

Size	Men's Shoe	Women's Shoe
XSmall	6 - 7	7 - 8
Small	8 - 9	9 - 10
Medium	9 - 11	10 - 12
Large	11 - 13	12 - 14
X-Large	13 - 14	—

800-327-4110



Website: **www.flaorthopedics.com**

©2004 FLA ORTHOPEDICS, INC.