

McDAVID™ ANKLE GUARD


WITH OPTIONAL PLASTIC INSERTS

Model No: 40-411

Sizes: XX Small - X Large
Color: White



With Optional Plastic Support Inserts

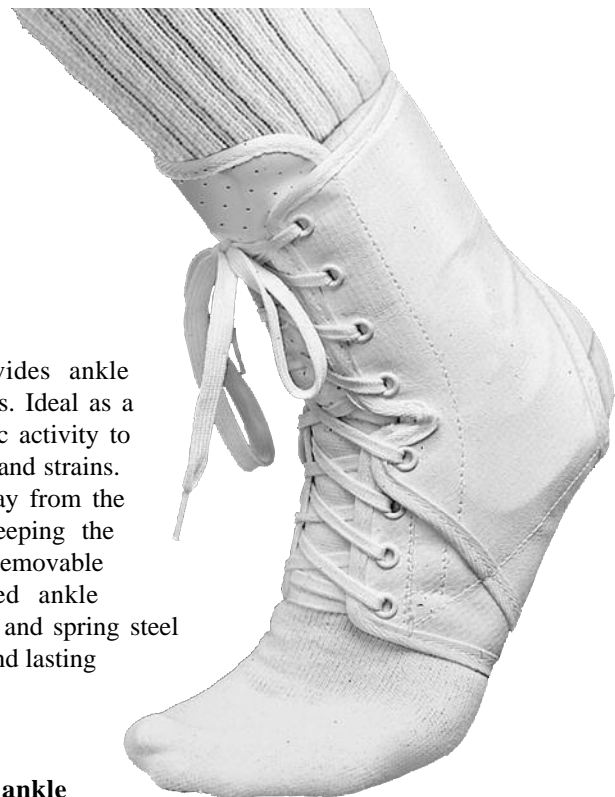

INDICATIONS:

- Instability
- Athletic Activity
- Moderate Sprains
- Post Surgery Ankle Weakness
- Injury or Reinjury Prevention

BENEFITS:

This lace-up ankle guard provides ankle stability to weak or injured ankles. Ideal as a preventative device during athletic activity to help reduce the chance of sprains and strains. Intera® lining draws moisture away from the skin for quicker evaporation, keeping the ankle cool, dry and comfortable. Removable plastic inserts provide increased ankle stability. Three layers of material and spring steel stays ensure solid ankle stability and lasting durability.

FEATURES:

- **Maximum, lightweight ankle support**
- **Soft, pliable and durable material** with lace-up front for a conforming and intimate fit.
- **Elastic insets** for increased support without limiting flexibility.
- **Notched front** allows the foot to flex freely.
- **Extended elastic heel** combines a positive heel lock and fit with no pinching of the Achilles Tendon.
- **Sewn-in Arch** adds comfortable firm-fitting support
- **Fits Left or Right**

HOW TO APPLY:

Wear brace over socks. Slip foot into brace and tighten laces from bottom to top. Retighten for an even and comfortable fit.

CARE INSTRUCTIONS:

Machine wash gentle cycle in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Size according to US shoe size.

Size	Men's Shoe	Women's Shoe
XX-Small	Under 6	Under 7
X-Small	6 - 7	7 - 8
Small	8 - 9	9 - 10
Medium	9 - 11	10 - 12
Large	11 - 13	12 - 14
X-Large	14 and Over	—

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.