

PROLITE® WRAP AROUND KNIT ANKLE SUPPORT

Model No: 40-500

Sizes: Small - XLarge

Color: Blue or Beige

Provides excellent compression and stabilization for weak or injured ankles. Stretch knitted Lycra/nylon material is a lightweight, breathable alternative to neoprene. The double woven elastic strap offers adjustable compression to both sides of the ankle and arch of the foot. The wide strap simulates taping and wraps in a figure-eight design.

- **KNIT DESIGN**

Unique design lays more stretch yarns where greater flexibility is desired - the result is increased comfort with precise compression and stabilization of the soft tissues while pressure is relieved from the actual tender joint.

- **STRETCH TOP BAND**

Comfortably keeps support in place Without digging into skin.

- **CONTOURED SHAPE**

Provides more precise compression and fit.

- **HEEL POCKET**

Provides a comfortable fit for comfortable, long term wear.

- **WIDE, WOVEN DOUBLE ELASTIC STRAP**

Wraps around the joint for increased stability. Figure-eight strap design simulates taping of the ankle.

- **LATEX FREE**

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

WEARING INSTRUCTIONS: Simply slip the support over ankle and pull into place. Support should fit snugly but not so tight that it digs into skin. Pull one elastic strap up over the foot. Wrap the strap around the back of ankle and close on itself using the hook and loop. Repeat for other strap, wrapping in the opposite direction.

TO DETERMINE SIZE:

Measure the ankle just above the ankle bone.

Size	Fits Ankles
Small	7 to 8½"
Medium	8½ to 9½"
Large	9½ to 11"
X-Large	11 to 12"



800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.