

INNERLOK 8™ ANKLE BRACE

Model No: 40-511

Sizes: Small - XLarge


Color: Black

USE:

Ideal for sports and to support weak or injured ankles.

FEATURES:

- **Internal Figure-Eight Straps** – Preconfigured in the correct position to assure proper strap placement every time. Provide extra support and allow for a more custom fit.
- **Short Curved Seamless Arch** – Fits the contour of the foot and allows full range of motion comfortably. Seamless arch virtually eliminates irritation to the bottom of the foot.
- **Breathable Tongue** – The advanced tongue material wicks moisture away from the skin and allows air to pass through to the skin, keeping the ankle cooler and drier. Tongue is completely sew in to stop migration and prevent pinch points.
- **Full Elastic Back** – Has elastic all the way up the back of the brace to ensure complete unrestricted blood flow to the Achilles tendon and eliminate the chance for blistering.
- **Junior Ballistic Nylon Construction** – Internal foam padding is laminated to a junior ballistic nylon exterior. The foam padding interior provides a comfortable fit while the junior ballistic nylon exterior makes the brace extremely durable. Materials are lightweight and fit easily in your regular athletic shoe.
- **Close Spaced Eyelets** – Three close spaced eyelets are located in the middle of the brace, directly opposite the heel opening, to lock the heel in place. This keeps the brace in the correct position and prevents slippage on your ankle.

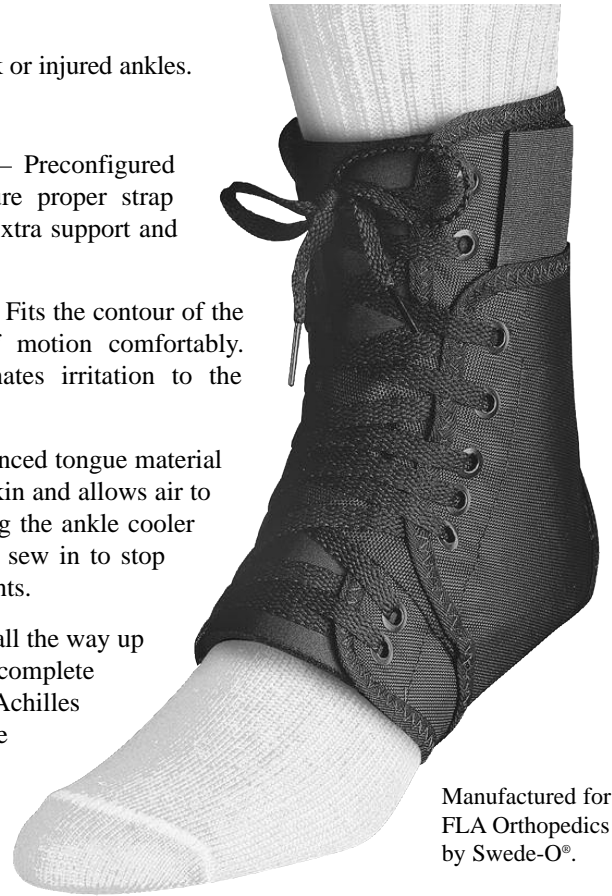

Manufactured for
FLA Orthopedics
by Swede-O®.

HOW TO APPLY:

Wear brace over sock. Slip foot into brace. Tighten the InnerLOK 8™ straps for a secure fit. First, adjust the inside strap until snug and repeat with the outside strap. Tighten laces from bottom to top. Retighten InnerLOK 8™ straps for an even and comfortable fit.

CARE INSTRUCTIONS:

Ensure straps are closed. Machine wash gentle cycle or hand wash in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Size according to US shoe size.

Size	Men's Shoe	Women's Shoe
Small	6 - 7	7 - 9
Medium	8 - 10	10 - 11
Large	11 - 12	12 - 13
X-Large	13 - 14	14+

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.