

*For Men  
& Women*



*Heats quickly in  
microwave!*

## Therall™ Therapeutic Foot Warmers

---

- Ideal for tired, aching or cold feet
- Relieves pain from arthritis or tendonitis
- Penetrating, soothing and long-lasting warmth
- Cozy fleece liner
- Premium, natural grain distributes heat evenly and conforms to the shape of the feet for effective pain relief
- Safe, effective and easy to use
- No fussing with cords or plugs
- Can be reheated over and over in microwave

**Use:** Provides soothing warmth and pain relief to tired, aching feet.  
*Ideal for arthritis, tendonitis, stiffness, chronic cold feet and sore or aching feet.*

Sizing: Size according to U.S. shoe size.

Size	Small	Medium	Large	X-Large
Men's	7 - 8	9 - 10	11 - 12	13 - 14
Women's	5 - 6	7 - 8	9 - 10	11 - 12

One Pair

Microwavable Slippers

# Therall™ Therapeutic Foot Warmers

---

**Heating Instructions:** Heat one Foot Warmer at a time in the microwave. **Heat on high for a maximum of 60 to 90 seconds** (depending on microwave's intensity).

<u>Microwave setting</u>	<u>Heating time</u>
High	60-90 seconds
<i>Heat one Foot Warmer at a time.</i>	
<b>DO NOT OVER HEAT.</b>	

Do not heat longer than recommended as this may burn the slippers. Always test the heat before wearing by placing hand in each Foot Warmer; ensure the temperature is comfortable. Must be worn with absorbent socks. Must cool to room temperature before reheating.

**Care Instructions:** DO NOT wash or submerge in water. Spot clean the surface with a mild soap and damp cloth. Socks must be worn with the slippers to keep the inside of product clean.

**Caution: Not to be worn while walking. Not suitable for persons with diabetes or anyone with decreased sensitivity, swelling of the feet or ankles or decreased circulation. Persons with these conditions should consult a physician before use. Should not be worn in place of regular slippers. The Foot Warmers are not designed for outdoor use.**

**Non-toxic**

**Latex Free**

Made in China • Manufactured for FLA Orthopedics, Inc.

©2007 FLA Orthopedics, Inc., Miramar, FL 33025

Part Series 53-425 • [www.flaorthopedics.com](http://www.flaorthopedics.com)