

SOFT FORM® HERNIA BRIEF

Model No: 67-500

Sizes: Small - XXLarge
Color: White

US Patents 6,622,719B1
& D482,512

IMPORTANT INFORMATION

Use this product only under the supervision of your physician and only for REDUCIBLE hernias (can be pushed back). A hernia should be inspected daily to ensure it is not becoming infected or developing into a more serious non-reducible hernia. Your hernia brief should be removed and the area checked frequently.

A non-reducible hernia can become life threatening if part of the intestine gets trapped or "strangulated" in the opening. This can lead to dangerous complications such as obstruction of the flow of intestinal contents or blood, leading to tissue death and gangrene. The symptoms of intestinal obstruction are nausea, vomiting, loss of appetite, or increased abdominal pain.

A strangulated hernia is an emergency situation that usually require immediate surgery.

The Soft Form® Hernia Brief offers gentle relief from reducible inguinal hernias by providing adjustable and comfortable compression. Low in profile and white in color, it can be worn inconspicuously under most clothing. May be worn in lieu of a regular undergarment and, with hernia support built into the brief, eliminates the need to wear both a regular undergarment and a bulky truss.



FEATURES & BENEFITS:

- **Built-In Hernia Support** - Hernia pad pockets are sewn into the brief and eliminate the need to wear both an undergarment and a truss.
- **Adjustable Compression Straps** - Hook and loop straps are easily adjusted for a comfortable, customized fit. The short, narrow straps create the ideal amount of focused compression without bulk or binding.
- **Flexible, Removable Foam Cushions** provide constant, gentle and comfortable compression on the weakened muscles, reducing the hernia by pushing it back. Brief includes a right and left pad and either pad can be removed to treat a single hernia.
- **Soft, Comfortable Materials** - Made with a soft and cool polyester/spandex fabric. Plush waistband is also extremely soft and allows for comfortable, extended wear without pinching the skin.
- **Latex Free**

APPLICATION INSTRUCTIONS:

The Hernia Brief includes removable left and right flexible foam pads. When inserted correctly, the rounded (curved) side of the pad goes against the body with the flat side facing outward. If treating a single hernia, remove the unnecessary pad from the support.

1. Unhook the end of elastic tension straps from the loop patches.
2. Step into and pull the brief up with the pads in the front (curved side against the body).
3. Pull the elastic tension straps to obtain the desired amount of compression and attach hook end of tension straps to loop patches.

This Hernia Brief can be worn in place of your regular undergarment.



Size	Fits Waist
Small	28 - 30"
Medium	32 - 34"
Large	36 - 38"
XLarge	40 - 42"
XXLarge	44 - 46"

Measure around waist to determine size.

CARE INSTRUCTIONS:

Remove foam pads. Fasten all closures. Hand wash in cool water with mild soap. Pat dry to remove excess water. Air dry away from heat. Wash as needed. *To ensure good hygiene, it is recommended that the Hernia Brief be washed as often as would any other undergarment.*

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.