

NEOPRENE HINGED KNEE STABILIZING BRACE

Model No: 37-104

Sizes: XSmall - XXXLarge

Color: Navy or Black

INDICATIONS:

- ACL Sprains and Tears
- Medial/Lateral Meniscal Tears
- LCL Sprains and Tears
- PCL Sprains and Tears
- MCL Sprains and Tears

BENEFITS:

This stabilizing brace allows for full flexion while protecting injured medial and lateral structures such as the MCL, PCL, medial meniscus and lateral meniscus via medial/lateral steel reinforcements. This stabilization prevents further stress which helps promote healing. Patellar malalignment is controlled by a superior horseshoe. The hinges also protect the ACL and PCL by preventing hyperextension.

PRODUCT FEATURES:

- Four way stretch for proper fit and comfort
- Sports neoprene for therapeutic warmth
- Superior horseshoe support
- Metal medial/lateral hinges in neoprene covered pockets
- Adjustable support straps
- Easy slip-on style

APPLICATION:

Insert foot into knee support and slide up over the knee. For proper fit, the top of the support should be positioned approximately four inches above the center of the knee. Hinge axis should be aligned with the center of the knee. Adjust the support straps so they fit comfortably around the knee without inhibiting circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap before wearing. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Measure 4" above the center of the kneecap.

Size	Fits Knees
XSmall	12 - 13"
Small	14 - 15"
Medium	16 - 17"
Large	18 - 19"
XLarge	20 - 21"
XXLarge	22 - 23"
XXXLarge	24 - 25"

WARNING:

Contact of this material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with this material. Anyone who develops skin irritation from contact with this material should discontinue its use immediately and consult a physician. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.



800-327-4110



www.flaothopedics.com

©2004 FLA ORTHOPEDICS, INC.