

SOFT FORM® WRAP- AROUND STABILIZING KNEE SUPPORT

Model No: 37-303

Sizes: XSmall - 3XLarge

Color: Beige

INDICATIONS:

- Slight to moderate ligament sprains and strains
- Tendinitis of the knee
- Arthritis
- Knee joint instability

FEATURES & BENEFITS:

- Unique Breathable Foam Material is latex free and neoprene free. Provides compression with a soft plush fabric against the skin. Non-irritating for those with sensitive skin.
- Wrap-Around Design is ideal for swollen or injured knees – quick and easy to apply.
- Flexible Spiral Stays provide medial and lateral (side-to-side) stability of the knee joint.
- Open Popliteal eliminates bunching and irritation behind the knee.
- Condyle Pads offer additional padding and stabilization to the patella.
- Adjustable Support Straps for an improved fit that stays in place.
- Latex free and Neoprene free.



APPLICATION INSTRUCTIONS:

Open adjustable straps and hook and loop closures on support. Position underneath leg so that popliteal opening in the back of the support is centered on the crease behind the knee. The medial and lateral stays should be aligned with the prominent bones on the each side of the knee. Secure the hook and loop thigh closure, then calf closure. Secure thigh and calf straps.


CARE INSTRUCTIONS:

Close straps. Hand wash in cold water with mild soap. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not use bleach.

HOW TO SIZE

Measure circumference of leg,
4" above center of the kneecap.

SIZE	FITS
X-Small	12-13"
Small	14-15"
Medium	16-17"
Large	18-19"
X-Large	20-21"
XX-Large	22-23"
XXX-Large	24-25"

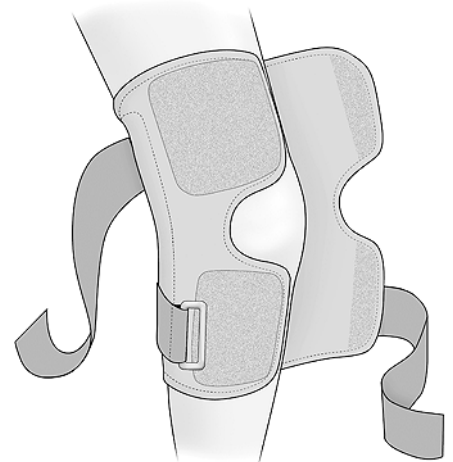

800-327-4110



www.flaorthopedics.com

©2007 FLA ORTHOPEDICS, INC.