

LUMBAR SACRAL SUPPORT WITH ABDOMINAL BELT

Model No: 31-208

Sizes:

Universal, Small - 4X-Large

Color:

White

SIZING:

Measure around hips just below the waistline.

Size	Fits
Universal	33 - 48"
Small	28 - 32"
Medium	33 - 36"
Large	37 - 40"
X-Large	41 - 44"
XX-Large	45 - 48"
3X-Large	49 - 52"
4X-Large	53 - 56"

USE:

Sufferers from occasional lower back pain resulting from muscle strain, fatigue, or injury may find relief by stabilizing the lumbar sacral and abdominal areas. If pain persists, consult a physician.

FEATURES & BENEFITS:

- High quality 10" wide elastic.
- Easy hook and loop closures adjust for the most comfortable and easy fit.
- Uniquely designed with an overlapping abdominal belt to provide additional lift and support in the abdomen area.
- Tapered front and V-shaped back allow for a comfortable fit.
- Six flexible stays in the back and sides provide strength and prevent the support from rolling.
- Lightweight and can be worn under most clothing.



WEARING INSTRUCTIONS:

Apply in standing position. Unfasten all closures. Place around back and upper buttocks area. Pull side with hook and loop fastener until the support holds the upper buttocks snugly but does not bind the groin area when sitting. Fasten closure on the support base. Pull the overlapping abdominal belt until snug and close hook and loop fastener.

CARE INSTRUCTIONS:

Fasten all closures. Hand wash in cold water with mild soap. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not machine wash or tumble dry. Do not bleach.



800-327-4110

Website: **www.flaorthopedics.com**

©1996 FLA ORTHOPEDICS, INC.