

9" NEOPRENE LUMBAR SACRAL SUPPORT

Model No: 31-722

Sizes: Small - 2X-Large

Color: Black

Indications:

- Occasional or Chronic Back Pain
- Muscle Strain
- Lumbar Sprain/Strain
- Sacral Contusion

SIZING:

Measure around hips just below the waistline.

Size	Fits
Small	28 - 32"
Medium	33 - 36"
Large	37 - 40"
X-Large	41 - 44"
XX-Large	45 - 48"

USE:

This support provides compression to the lumbar sacral and abdominal regions, stabilizing and supporting these areas to provide relief from lower back pain and discomfort.

FEATURES & BENEFITS:

- 9" high neoprene back panel provides therapeutic warmth.
- Tapered panels contour the body for excellent compression and support.
- Ventilated side panels for comfortable support.
- Flexible stays provide strength and stability to the lumbar region and prevent the belt from rolling.
- Low profile for inconspicuous wear underneath clothing.
- Easy application with simple hook and loop closures.
- Black color for extended wear.



WEARING INSTRUCTIONS:

Unfasten all closures. While standing, wrap the support around lower back. Center the stays in the back panel on the lower back. Pull side with hook fastener until desired compression is achieved and secure closure. Next, pull the overlapping neoprene straps to desired compression and secure. The support should fit snug but not too tight.

CARE INSTRUCTIONS:

Fasten all closures. Hand wash in cold water with mild soap. Wrap in towel to remove excess water (do not twist or wring). Air dry away from heat. Do not machine wash or tumble dry. Do not bleach.

Consult a physician if pain, swelling or unusual reactions are experienced. This device is only one element in the overall treatment program. There is no guarantee that injury will be cured or prevented through use of this product.



800-327-4110

Website: **www.flaorthopedics.com**

©2004 FLA ORTHOPEDICS, INC.