

SACRAL ILIAC SUPPORT WITH COMPRESSION PAD

Model No: 31-801

Sizes: Universal,
Small - XX-Large

Color: White

Indications:

- Sacral Iliac pain
- Sacral strain

Features & Benefits:

- Six inch high support provides compression to the sacral iliac region.
- Has a removable stockinette covered foam compression pad with a rigid back.
- Contoured in front to fit around the hips.
- All elastic support with adjustable hook and loop closure.



Wearing Instructions: Apply in a standing position. Unfasten hook and loop closure. Insert compression pad into pocket in back of support. Place support around lower back and upper buttocks area with pocket opening facing upwards. Pull side with hook fastener until the support holds the upper buttocks snugly. Secure hook to loop on the support base. Compression pad should be resting on lower back.

Care Instructions: Fasten closure and hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach. Wash as necessary.

TO DETERMINE SIZE:

*Measure around hips just below
waistline.*

Size	Fits
Universal	33 - 48"
Small	28 - 32"
Medium	33 - 36"
Large	37 - 40"
X Large	41 - 44"
XX Large	45 - 48"

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.