

PERFORATED SUEDE FINISH WRIST SPLINT

Model No: 22-442

8" Length

Sizes: Pediatric - XLarge

Model No: 22-441

10" Length

Sizes: Small - XLarge

Color: Beige



Latex-Free

*Non-Retail
Packaged*

INDICATIONS:

- Moderate/Severe Wrist Sprain
- Moderate/Severe Wrist Strain
- After Cast Removal
- Minor Fracture
- Advanced Carpal Tunnel Syndrome

BENEFITS:

Recommended when greater immobilization is needed. Supports and stabilizes the wrist following minor fractures, sprains, or after cast removal. Allows for full finger dexterity, although the rigidity of the brace reduces strong gripping.

PRODUCT FEATURES:

- Maximum immobilization provided by the wide contoured palmar stay and the two dorsal stays
- Made of perforated suede finish material that is durable, cool and comfortable
- Soft padded liner absorbs moisture
- V-shaped strap disperses the tension over a wider area
- Latex-Free

HOW TO APPLY:

Place on hand with the palmar stay on the palm and the back open. Secure body of support around the wrist with hook and loop closure. Pull thumb strap across and secure. Wrap each support strap around forearm and secure with hook and loop closure.

CARE INSTRUCTIONS:

Fasten closures and hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING:

Measure around center of wrist. Specify Right or Left.

Size	Fits Wrists
Pediatric	Under 4 1/2"
Small	4 1/2-5 1/2"
Medium	5 1/2-6 1/2"
Large	6 1/2-7 1/2"
XLarge	Over 7 1/2"

800-327-4110