

UNI-FIT® UNIVERSAL WRIST SPLINT

Model No: 22-602

Sizes: Universal

Color: Black

US Patent D477,088

USE:

Ideal brace when stabilization is needed for weak or injured wrists, wrist strains, wrist sprains, Carpal Tunnel Syndrome, or after cast removal. Palmar and dorsal stays prevent wrist flexion/extension and conform to the shape of the wrist to maintain a neutral position. Removable dorsal stays can be placed properly or removed completely allowing for a less rigid feel. Ideal material for those sensitive to latex or heat build-up. 8" length.



COMFORT BENEFITS:

- **Universal design** fits left or right hand and a range of sizes.
- **Removable dorsal stays** can adjust for proper fit or decrease the degree of immobilization
- **Top strap** is angled downward and set back for a more conforming fit without bunching.
- **Inner sleeve** holds the splint in place for easier application.
- **Loop lock closures** ensure proper fit and easy application.
- **Rigid stays** maintain wrist in neutral position and prevent flexion and extension of the wrist.
- **Latex Free**

WEARING INSTRUCTIONS:

Unfasten closures. Slip wrist under elastic insert and through the thumb hole, making sure the removable stays are located on top of the wrist. Adjust removable dorsal stays to proper position. Secure straps around wrist and then secure top strap. If overlapping is necessary to prevent bunching, be sure to place the side with the top strap underneath the dorsal side.) Straps can be trimmed to remove excess length. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water and mild soap. Wrap in towel to remove excess water. Do not twist or wring. Air dry away from heat. Do not tumble dry. Do not bleach.

SIZING: Measure around the center of wrist.

Size	Fits Wrist
Universal	5 - 11"

800-327-4110



Website: www.flaorthopedics.com

©2002 FLA ORTHOPEDICS, INC.